**PLEASE CONTACT MOEN FIRST**

For Installation Help, Missing or Replacement Parts

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1-800-465-6130

WWW.MOEN.CA

Installation Instructions For Grab Bars with Press and Mark

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(Save instruction sheet for future reference)

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www.moen.com/product-registration

Thank you for purchasing this Moen® product. Moen is a leader in grab bars, offering top-quality products that are attractive and innovative, with performance features like a 500 lb. weight pull capacity that exceeds ADA requirements.

The entire Moen family of products offers people new options for independent living by providing outstanding comfort, convenience and style.

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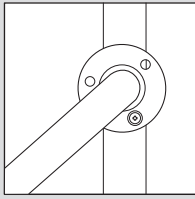
All Moen grab bars feature the exclusive patented SecureMount™ mounting flange.

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Conventional Mounting / Montage conventionnel / Montaje convencional


Conventional mounting configurations often only allow for one or two screw connections to the stud, making installation difficult and less secure.

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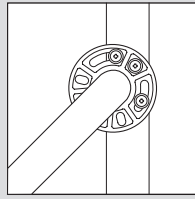SecureMount™ / SecureMount™^{MC} / SecureMount™

The patented SecureMount™ design makes grab bar installation easier and more secure with an innovative design that features multiple slots for screws, allowing secure connection with three screws into the wood stud at any angle.

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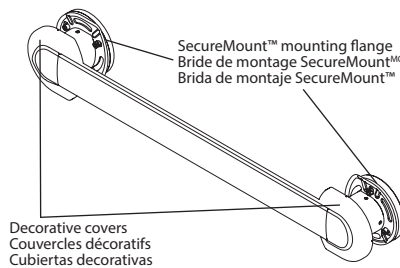
Components included / Pièces incluses / Componentes incluidos


Grab bar, with SecureMount™ mounting flanges and decorative covers

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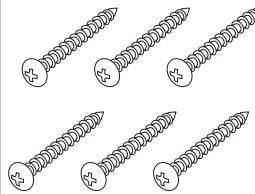
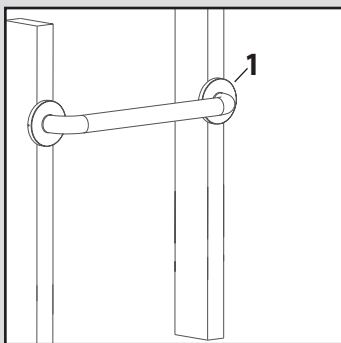
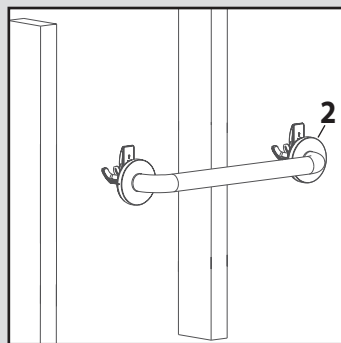
2. Hollow Wall Installation with two SecureMount™ Anchors

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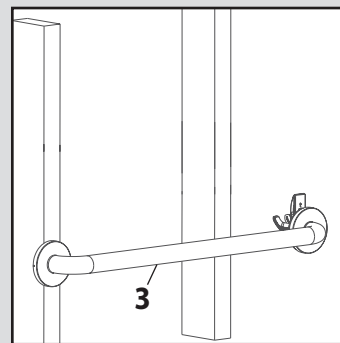3. One side mounted to stud and one side mounted in hollow wall with SecureMount™ Anchor

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IMPORTANT:

Please note that screws MUST be mounted into a wood stud for secure installation.

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1

Decide whether you would like to position your grab bar horizontally, vertically or at an angle, remembering that BOTH ends of the grab bar MUST be positioned over a wood stud (unless using SecureMount™ anchors).

Note: Diagonal/angled installation is possible only with grab bars 24 in. or longer.

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2

Locate the studs behind the wall by using a stud finder or other stud locating method. Stud spacing is usually 16 in. apart; locate the edges of each stud. Screws to be installed per Figure 2, based on installation orientation.

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Press and Mark Installation Instructions

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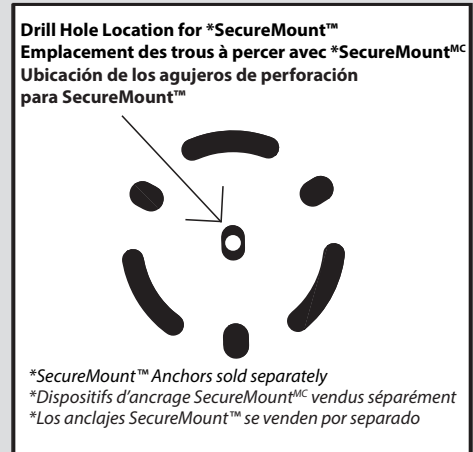
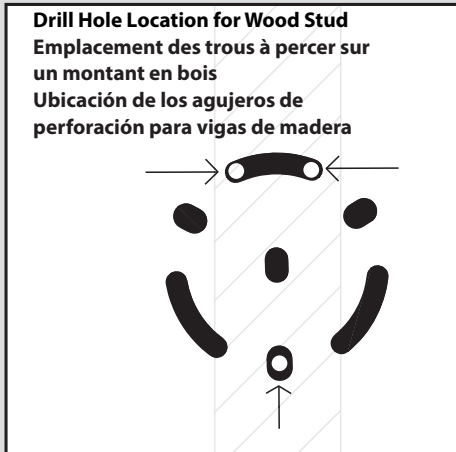
Wood vs. SecureMount™ Press and Mark Drill Location

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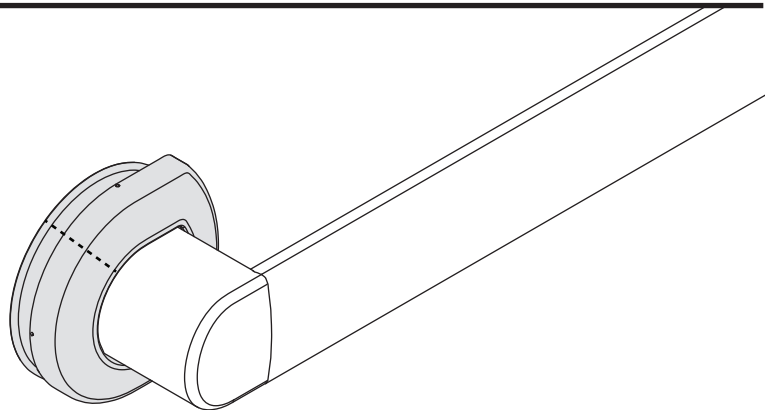
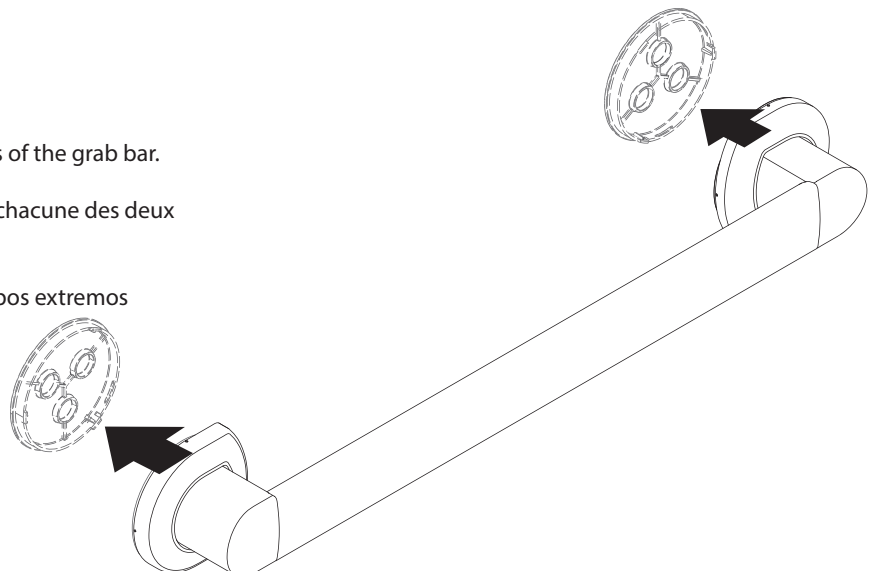
5 Press & Mark

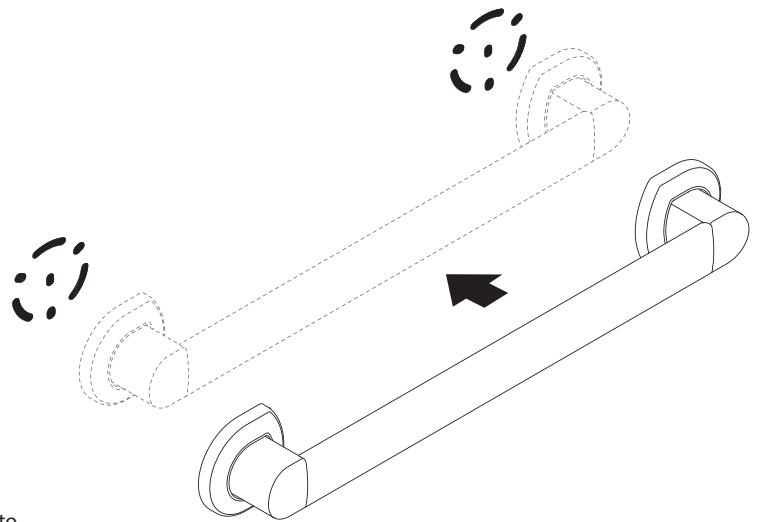
Align grab bar in desired location, press & mark by pushing both Posts firmly and evenly against the wall. Hold for a few seconds, then set assembled product aside. You'll now see marks showing you exactly where to drill.

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6 Verify

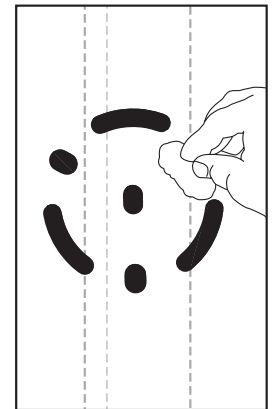
If you don't like the marking on the wall, wipe the wall clean using a damp cloth and re-mark the location.

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7

Slide decorative covers (x2) towards the grab bar.

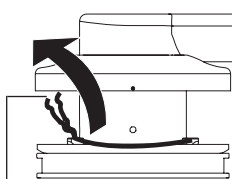
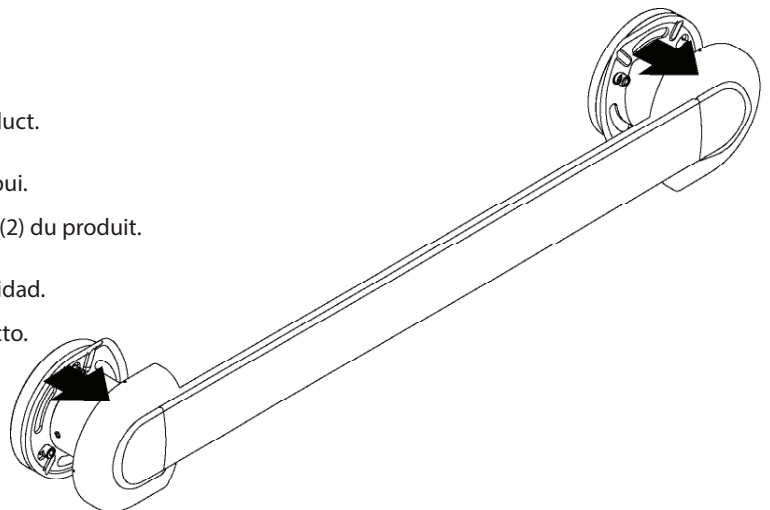
Remove twist ties (x2) and ink stampers (x2) from the product.

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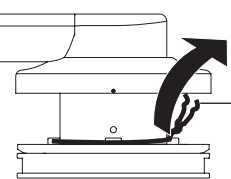
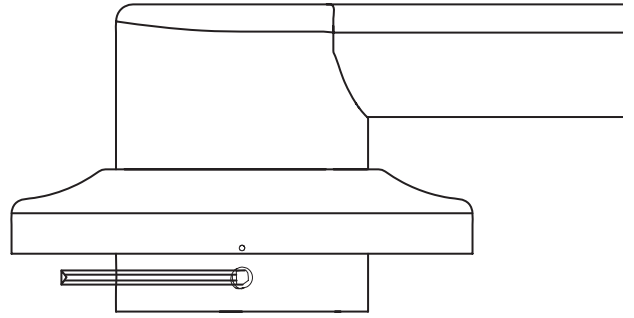
8

For easier installation and mounting remove 3 set screws from each flange and remove bar from SecureMount™ flange.

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
9

Remove cover and re-assemble flange to bar by tightening 3 set screws.

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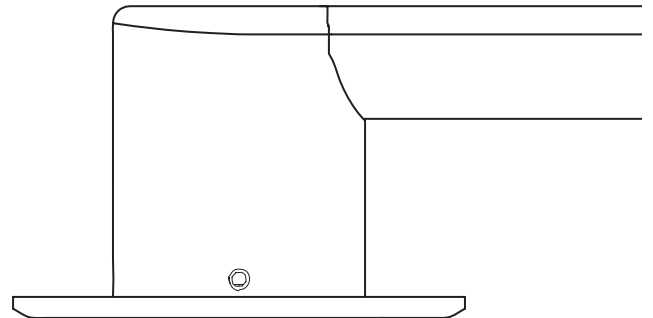
For Drywall installation using SecureMount™ Anchors, refer to INS10306.

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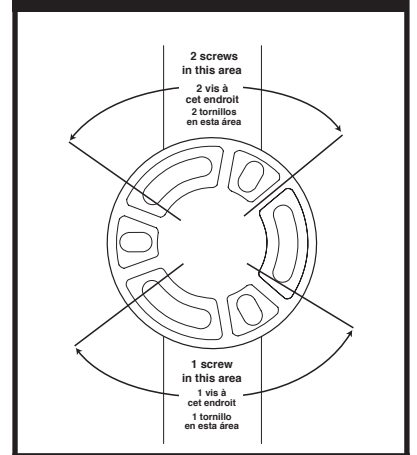FIGURE #3: / FIGURE 3: / FIGURA 3 :

10

Drill 1/8" diameter pilot holes at each screw location. Make sure to drill into the stud. Refer to Figure 3 for hole locations.

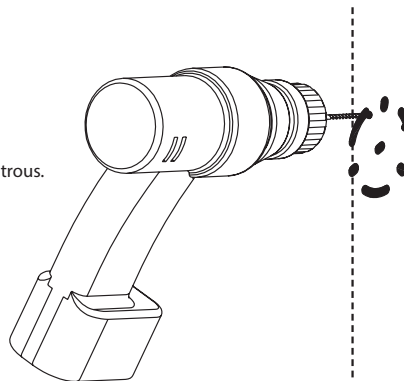
NOTE: For easier screw installation, drill pilot hole the length of the screw.

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11

TIP: To prevent damage to the bar, use 3" long #2 Phillips Bit

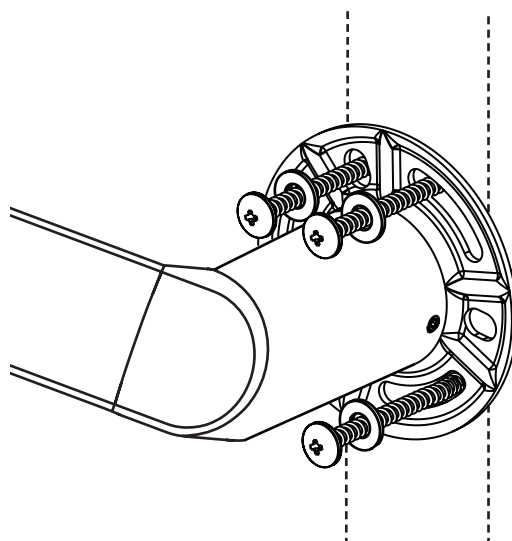
If desired, apply silicone caulk to the back of the SecureMount™ plate to guard against water penetration. Install 3 screws and washers into each SecureMount™ flange and tighten until secure.

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
12

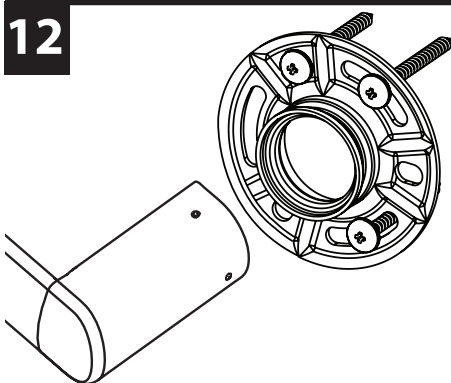
Dis-assemble bar from SecureMount™ flange by removing set screws.

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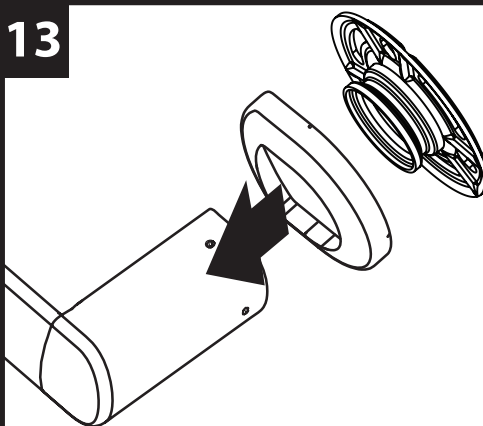
Install covers.
Placer les couvercles.
Instalar las cubiertas.

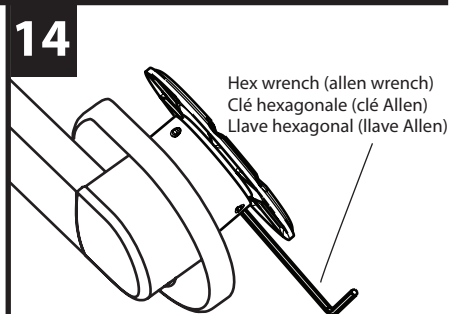14

Place bar over SecureMount™ flanges and tighten all set screws.

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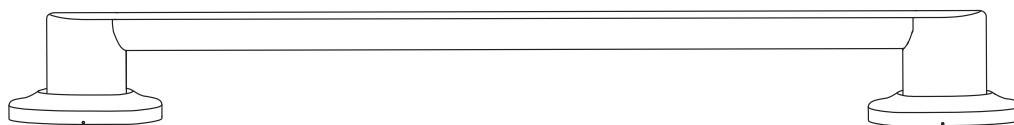
Enclencher couvercles.

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WARNING:

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 500 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact Moen immediately at 1-800-289-6636 if any components are missing or damaged. Install in accordance with Moen's instructions. Do not modify this product in any way. Check grab bar periodically after installation to be sure all screws are secure. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT MOEN AT 1-800-289-6636.

WARRANTY FOR MOEN HOME CARE BATH SAFETY PRODUCTS:

LIMITED LIFETIME WARRANTY

Moen warrants to the original consumer purchaser for as long as the original consumer purchaser owns their home (the "Warranty Period" for homeowners) if this product fails due to materials or workmanship at any time during the life of the product Moen will replace it free of charge. Purchasers for residential use where the occupant is not the owner of the property (apartments, duplexes, or other multi-residential unit housing) are warranted for a period of 10 years from the original date of purchase (the "Warranty Period" for multifamily owners). Purchasers for industrial, institutional and all other business uses are warranted for a period of 5 years from the original date of purchase (the "Warranty Period" for commercial owners.) Proof of purchase (original sales receipt) from the original consumer purchaser must accompany all warranty claims. Contact Moen at 1-800-289-6636 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. **TO THE EXTENT PERMITTED BY LAW, MOEN DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.**

LIMITATION OF LIABILITY:

THIS WARRANTY IS YOUR SOLE REMEDY AND MOEN SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL MOEN'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.moen.com

Moen Grab Bars meet or exceed the applicable requirements of the following standards/guidelines: The Americans with Disabilities Act (ADA).

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Moen Incorporated ©2019 • www.moen.com